



# #65120

EN

## Instructions for use of the One® Single-Use XL Guarded cover #17206DXXX

### Indication for use

The Moria ONE® Single-Use XL Guarded cover is designed for performing lamellar keratoplasty. The One® Single-Use XL Guarded cover #17206DXXX is used after using the One® Single-Use Guarded Punch #17207DXXX.

### Device components

- Trephine blade housing (see Fig. 1)

To be used with a cutting block base (A) (see Fig. 2), a syringe (C) (see Fig. 2) and a clamp (D) (see Fig. 2), provided separately with the One® Single-Use Guarded Punch #17207DXXX.

The most recent version of this document is available on Moria website (<http://www.moria-surgical.com>) via the restricted access heading «Documentation/Library». Translations are also available in other languages under that same heading. Registration required.












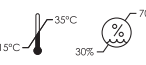



### Warning

- These instructions explain only how to use this medical device and do not describe the steps necessary to perform a corneal graft. The surgical procedure remains the responsibility of the surgeon experienced in performing corneal grafts.
- Check the integrity of the pouch.
- Check also the expiration date on the packaging.
- The ONE® Single-Use XL guarded cover is sterile and ready for use. Do not use it if the pouch in which it is packaged is damaged.
- Design of this medical device does not allow its reuse. Reusing this medical device affects its clinical performances and exposes the patient to adverse events.

### Directions

- 1 / Check the blade diameter which is stamped on top of the trephine blade housing.
- 2 / Remove the sterile blister from the pouch.
- 3 / Remove the trephine blade housing from the blister.
- 4 / Take the cutting block base and the syringe previously connected with the Luer™ lock (components provided separately with the One® Single-Use Guarded Punch #17207DXXX) with the graft on it and already trephined.
- 5 / Check the integrity of the trephine blade under the microscope.
- 6 / Place the trephine blade housing on the cutting block base, let it lower down by gravity to avoid setting it with a wrong angle (risk of blockage) (Fig. 4) then press down gently to cut the graft (Fig. 5) **Warning : Do not turn the punch during or after trephination.** While holding the assembled punch, grasp the sclera, through the "top lateral" window, with a tooth forceps then gently lift the trephine blade housing up (Fig. 6).
- 7 / Pull gently upward the trephine housing blade from the cutting block base. The donor button should be kept in place on the cutting block base thanks to the aspiration. Then remove completely the trephine housing blade.
- 8 / Disassemble the syringe from the tubing to release the vacuum. The donor button is now ready to be used on the patient.
- 9 / At the end of the surgical procedure, discard the ONE® Single-Use XL Guarded Punch into an appropriate container for destruction (according to the regulations in force).

## Explanation of symbols

 xxxxxx	CATALOGUE REFERENCE
	QUANTITY
 YYYY/MM/DD	USE BY
 xxxxxx	BATCH CODE
	STERILIZED USING ETHYLENE OXIDE
	DO NOT REUSE
 xxxxxxxx	MANUFACTURER
 YYYY/MM/DD	DATE OF MANUFACTURE
 xxxxxx	CAUTION: CONSULT ACCOMPANYING DOCUMENT(S)
 xxxxxx	CONSULT OPERATING INSTRUCTIONS
	DO NOT USE IF PACKAGE IS DAMAGED
 15°C 35°C 30% 70%	STORAGE CONDITIONS: <ul style="list-style-type: none"> <li>• TEMPERATURE: 15°C – 35°C / 59°F – 95°F</li> <li>• HUMIDITY RATE: 30% – 70%</li> </ul>
	KEEP DRY
	DISCARD IN A SEPARATE COLLECTION BIN
	CAUTION FOR USA ONLY: US FEDERAL LAWS RESTRICT THIS DEVICE TO SALE BY, OR ON THE ORDER OF, A PHYSICIAN.

## Reference

Reference	#17206DXXX
Designation	Single-Use XL Guarded cover DXXX
Packaging	1 per box, sterile, single-use

**Fig.1**


#17206DXXX contains the following element



← Trephine blade housing

**Fig.2**

To be used with #17207DXXX, which contains the following elements:

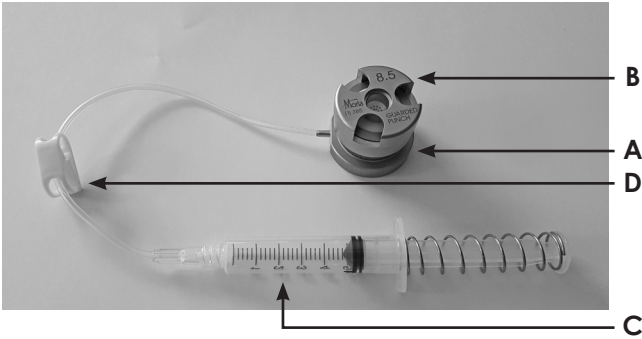

<b>A</b>	Cutting block base
<b>B</b>	Trephine blade housing
<b>C</b>	Syringe
<b>D</b>	Clamp


**Fig.3**




**Fig.4**



**Fig.5**

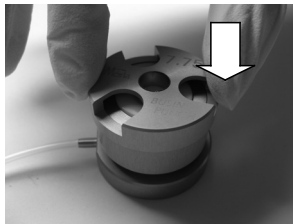

**Fig.6**

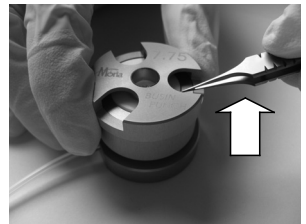

**To obtain more information**

**MORIA SA**

15, rue Georges Besse  
92160 Antony

FRANCE

Phone: +33 (0) 1 46 74 46 74

Fax: +33 (0) 1 46 74 46 70

[www.moria-surgical.com](http://www.moria-surgical.com)



**MORIA Inc**

1050 Cross Keys Drive  
Doylestown, PA 18902

USA

Phone: (800) 441 1314

Fax: + 1 (215) 230 7670

[www.moria-surgical.com](http://www.moria-surgical.com)

**Moria Japan K.K.**

Arcadia Building 6F  
1-12-3 Kanda  
SudachoChiyoda-Ku  
Tokyo 101-0041

JAPAN

Phone: 81-3-6260-8309

Fax: 81-3-6260-8310

[www.moriajapan.com](http://www.moriajapan.com)

**Moria Shanghai Office**

RM4, 10 FL, Kai Li Building  
N°432 Huai Hai West Road  
200052 Shanghai

CHINA

Phone: +86 21 52585278

Fax: +86 21 52585278

[www.moria-surgical.com](http://www.moria-surgical.com)

