One Use-Plus

EN


Information sheet for the setting of the OUP single-use head

OUP ring labelling


OUP SYSTEM


- OUP heads (box sticker of blue color)
- OUP suction ring (OUP labelling and stop of blue or yellow color)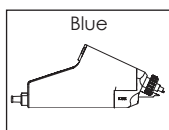

Ref. : 19345

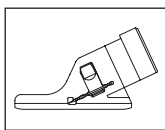Ref. : 19393/XX

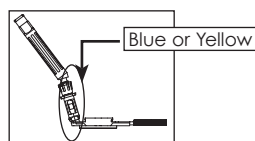Ref. Blue
19391/-1
19391/0

Ref. Yellow
19391/1
19391/2
19391/3

Warning

Do not use Epi-K™ system with OUP components

Contents

One Use-Plus single-use head with blade. **Fig.1**

Directions

A Assemble the head on the handpiece

- 1 / Remove inner plastic pack from outer peel pouch.
- 2 / Place the pack with head screw up **Fig.2A** and open top of the pack **Fig.2B**.
- 3 / Screw the locking screw clockwise to fix the head on the handpiece. Check that the head is screwed tightly to the handpiece. There should be no rotation of the head **Fig.3/4/5**.
- 4 / Once assembled, inspect blade under the microscope.
- 5 / Lubricate the blade prior to use with BSS® (Trade mark of Alcon) or another appropriate solution.
- 6 / Activate the handpiece to check if the blade is engaged and oscillates properly.


B Assemble the handpiece on the suction ring

- 1 / When inserting the head in the rails of the suction ring, the handpiece should be at a 45° angle from the metal threaded shaft of the ring to prevent the shaft from touching the head and separator assembly **Fig.6**.
Once the head is well inserted in the rails of the suction ring, gently decrease the 45° angle to 0° and advance slightly the head into the ring **Fig.7**.

- 2 /  **WARNING! Do not touch the blade (as shown below in the picture) with the threaded shaft, it would damage the edge of the blade Fig.8.**

C Disassemble OUP single-use head

To disassemble OUP single-use head, insert it into the pack and unscrew the locking screw anticlockwise. Discard everything in a biohazard container **Fig.9/10/11**.

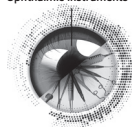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MORIA Inc